

**New
From SDI!**

Filtrette SYSTEM™

The Low-Cost Alternative to ndd's Spirette™!

Filtrette System – Better *Infection Protection* for Patients and Healthcare Professionals.

SDI's Filtrette System is the new, LOW COST alternative to the ndd Spirette. This two-piece system provides virtually perfect *infection protection* for both expiratory and inspiratory spirometry testing.

The Filtrette System incorporates a highly efficient single-patient filter that virtually eliminates cross-contamination from patient to patient *and* from patient to health care professional.

The FloTube section fits perfectly in ndd spirometers, and can be used more than 50 times before it needs to be replaced, making it both economical and convenient.

The one-time-use AstraGuard filter attaches securely to provide a bacterial/viral filter with virtually perfect infection protection.

Using the Filtrette System will not affect spirometer warranty.


Using the SDI Filtrette System:



1. Attach FloTube to ndd spirometer.



2. Insert single-use AstraGuard filter into FloTube.

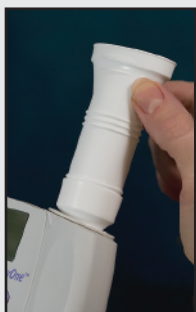

3. Spirometer is ready to use.





Filtrette System Features

- Lower cost
- Adapts to fit all ndd instruments
- Provides 99.9% bacterial and viral efficiency*
- Reduces risk of cross-contamination for patients and health care professionals
- Decreases time spent disinfecting equipment
- Offers low resistance to air flow
- Features small, non-threatening size
- Has a comfortable oval mouthpiece

The Filtrette System is available as a kit with The Klip, our noseclip.

Call your local distributor for details.

Cross-section of the filter shows placement of interior media. Edge of media is bonded to the exterior housing to form a tight seal.

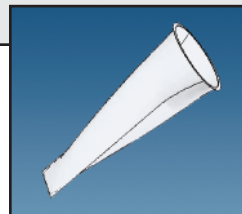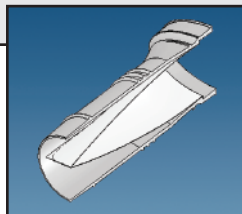

The filter media is tailored to fit inside every mouthpiece, providing an effective guard against cross-contamination.

Spirette is a trademark of ndd Medical Technologies.

* As tested by Nelson Laboratories, September, 2006.

Your dealer is:

sd**i**

DIAGNOSTICS

The Spirometry Source

Ten Hampden Drive, Easton, MA 02375

www.sdidiagnostics.com

© Copyright 2007